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**EN | A SIMPLE, EFFECTIVE, RISK-FREE HYGIENE ROUTINE**

**AUDISPRAY ADULT – 100% natural product**

AUDISPRAY ADULT contains a hypertonic seawater solution which is microbiologically controlled. AUDISPRAY ADULT avoids the use of cotton buds, which push the wax towards the eardrum.

**What is AUDISPRAY ADULT used for?**

AUDISPRAY ADULT softens and dissolves the wax, making it easier for it to be naturally evacuated from ear. Used regularly, AUDISPRAY ADULT prevents the accumulation of wax and impurities in the ear canal. AUDISPRAY ADULT, with no propellant gas and with an anti-overpressure nozzle, permits regular, effective and risk-free hygiene.

**Who can use AUDISPRAY ADULT?**

**Adults and children aged over 12 :** from 1 to 2 sprays in each ear, 2 to 3 times a week. **People who produce a lot of wax** can use AUDISPRAY ADULT every day. **Hearing aid wearers**, for whom the use of AUDISPRAY ADULT is recommended every night after removing the hearing aids, to prevent earwax from interfering with the function of the hearing aids.

**How to use your AUDISPRAY ADULT ?**

**1.** Point the nozzle towards the ear and administer 1 to 2 sprays per ear, keeping the bottle upright. **2.** Rub the base of the ear for a few seconds. **3.** Tilt your head to the side to let the surplus run out, and wipe dry. **4.** Rinse the nozzle with hot water after each use.

**Contraindications**

Do not use in cases of otitis, perforated eardrum or generally in any person with a medical history (vertigo, medical or surgical treatment, etc.)

**Precautions for use**

If you are not sure, or if you have pain before using the product, ask your pharmacist or doctor for advice. If a plug of earwax has already formed AUDISPRAY ADULT can cause the plug to swell, leading to discomfort. In this case, please consult your doctor. It is recommended that you use AUDISPRAY ADULT within a year after the first use of the bottle. Do not use the product after the expiry date stated on the packaging and on the bottle.

**AUDISPRAY ADULT is a medical device which conforms to European Directive 93/42/EEC | Tel : +41 22 718 64 20 | E-mail : uk@diepharmex.com**

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